

# Olympia Soft

**10843** Olympia Soft integrates care, comfort and elegance in one single bed. The low bed position of  $\pm 25$  cm reduces the risk of falling to an absolute minimum, whereas the ergonomic working height of  $\pm 80$  cm guarantees efficient care.

## Carriage

- Highly stable high/low adjustment by means of supporting arms.
- Reverse Trendelenburg adjustment up to  $\pm 16^\circ$ .
- Trendelenburg adjustment up to  $\pm 16^\circ$  (\*).
- Hidden castors  $\varnothing 50$  mm with synthetic cover caps, braked 2 per 2.
- Design pedal to activate the brake mechanism.
- The carriage has no actuators, which facilitates its accessibility and possibility to clean.
- The use of a patient lift is perfectly possible.
- The bed can be moved in every position, except in the safety position (= lowest position).

## Mattress platform

- Three- or four- (\*) section mattress platform.
- The backrest is electrically adjustable up to  $\pm 70^\circ$ , with auto-regression.
- The knee rest is electrically adjustable up to  $\pm 34^\circ$  (in case of 4 sections).
- The footrest is manually adjustable to obtain the Fowler position (in case of 4 sections).
- The legrest is manually adjustable up to  $\pm 14^\circ$  (in case of 3 sections).
- The electric manipulation of the mattress platform sections is equipped with a clamp safety.
- The mattress platform consists of rounded steel mattress platform profiles.
- The mattress platform is equipped with openings for patient restraint straps.
- The backrest and footrest are equipped with 2 synthetic mattress holders.
- The head end is equipped with 2 cases to install an IV rod or a lifting pole.
- No visible fastening bolts in the foot panel.
- The surrounds of the bed consist of 3 closed panel and a higher head panel, equipped with fire-retardant foam and upholstery.
- Round corners to increase the passive safety.

## Electric bed manipulations

- The electric functions can be activated by means of a hand switch.
- The hand switch is equipped with a preprogrammed sleeping function.
- The hand switch with selective locking system (\*) makes it possible to lock the bed functions selectively by means of a magnetic key.

## Paneltypes

Upholstered head panel.

## Accessories


We would like to refer to our accessory list.

(\*) Option



## Dimensions and technical data

- Width: 101 cm, Length: 208,5 cm
- Height variation of the mattress platform:  $\pm 25$  cm  $\pm 80$  cm
- Clearance height of the carriage: 150 mm (with a mattress platform height of 42 cm)
- Safe working weight: 230 kg
- Weight (without options): 140 kg
- Mattress platform dimensions: 204 x 86 cm
- Mattress dimensions: 195 x 85 cm, thickness: 12 to 15 cm

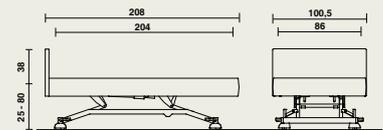

## Materials and finishing

- Metal: epoxy-coating RAL 9006 aluminium.
- Wood: OSB.
- Foam: CMHR- foam (fire-retardant).
- Self-lubricating synthetic hinges (= maintenance-free).
- Withstands the common cleaning products and disinfectants.

For more information about the **materials and construction**: please consult our material lists. We would like to refer to our extensive **colour gamut**.

## Options

- 4- section steel mattress platform.
- Mattress platform with removable synthetic sections.
- Illuminated hand switch (with 4-section mattress platform).
- Hand switch with selective locking system (with 4-section mattress platform).
- Infrared remote control.
- Battery.
- Reading lamp.
- Manual quick release (CPR).
- Central brake castors  $\varnothing 100$  mm activated by a brake bracket.

